



SANI-99™ FOR VETS

SANI-99™ FOR VETS CLINICAL DISINFECTION PROTOCOL

Multi-Surface Veterinary Grade Disinfectant


- Veterinary-grade, multi-surface disinfectant for clinical use
- Rapid, broad-spectrum efficacy against bacteria, viruses (including enveloped viruses such as coronavirus), and fungi
- Proven effective against key veterinary pathogens, including surrogate viruses for parvovirus, norovirus, and avian influenza
- Effective in both clean and dirty conditions
- Suitable for use across surfaces, equipment, and animal housing in clinical, agricultural, and animal care environments

Step One: Pre-Clean Surfaces



Remove all visible dirt, organic matter, and debris prior to disinfection using a suitable detergent. Pre-cleaning is essential to ensure effective disinfection.


Step Two: Prepare Solution

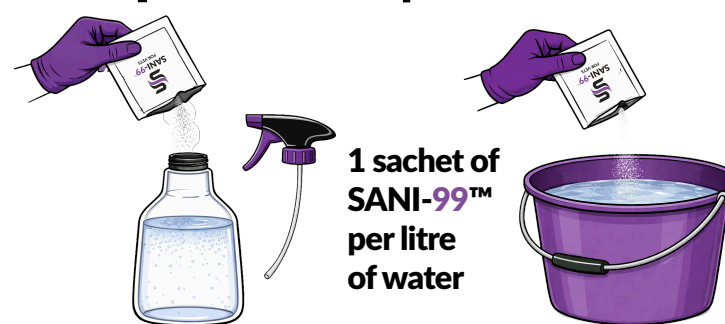

1 sachet of
SANI-99™
per litre
of water

Prepare solution by dissolving 1 x 6g sachet in 1 litre of clean water (spray bottle or bucket). Mix thoroughly until fully dissolved. Use 12g per litre in high-risk areas (e.g. parvovirus).

Step Three: Apply Disinfectant

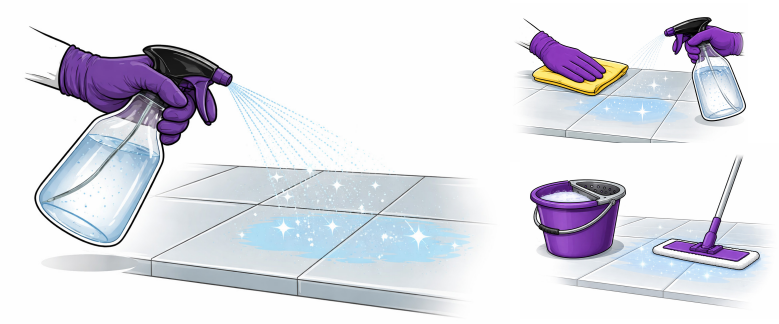

Apply evenly to the surface using a spray, cloth, or mop, ensuring full coverage (approximately 14 ml per m²) and that the surface remains visibly wet.

Step Four: Allow Contact Time

Contact Time Guide	
Application	Contact Time
Bacteria	10–30 seconds
Enveloped Viruses	Minimum 30 seconds – 5 min
Fungi	Up to 15 minutes

Ensure surface remains visibly wet. Do not wipe during contact time.

Kills bacteria
& viruses in
**10-30
seconds**




Allow the product to remain on the surface for the required contact time (e.g. bacteria: 10–30 seconds, enveloped viruses: minimum 30 seconds, fungi: up to 15 minutes). Do not wipe during this contact time. Allow to air dry or wipe after the required contact time if necessary.


Suitable for Use on:



Recommended Use:

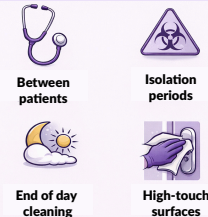

PPE Guidance

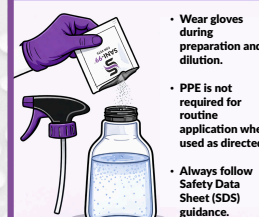

- Wear gloves during preparation and dilution.
- PPE is not required for routine application when used as directed.
- Always follow Safety Data Sheet (SDS) guidance.

For optimal performance, ensure surfaces are visibly clean prior to disinfection and that the prepared solution is applied to achieve full, even coverage with a visibly wet surface.

Allow the product to remain in contact for the required time (e.g. bacteria: 10–30 seconds, enveloped viruses: minimum 30 seconds, fungi: up to 15 minutes) and **do not wipe during this contact time**. Surfaces should be allowed to air dry or wiped after the required contact time if necessary.

Always follow all 4 steps to ensure effective disinfection.

Always follow label instructions and refer to the relevant Safety Data Sheet for handling, storage, and safety information.



For further information and MSDS contact: Samaxia Ltd.
Tel: 01522 454670 Email: enquiries@samaxia.com